











Navigating Incontinence Products for Your Adult Members!

At HCD, our team of experts ensures your members receive incontinence products tailored to their specific needs, providing optimal fit and protection. With our commitment to personalized care, we prioritize their comfort and confidence in managing their condition effectively. Here's how:

- ✓ **Right Product** – We put your members in the correct product based on the type of incontinence they are experiencing and the absorbency that is required.
- ✓ **Right Fit** – We ensure your members are receiving products in the correct size to prevent leaks and discomfort.
- ✓ **Right Amount** – We provide the appropriate quantity of products needed for your members to live with confidence and dignity.
- ✓ **Right Education** – We provide your members with information, so they understand how their products are supposed to fit and how often to change them. This improves product satisfaction and reduces misconceptions about product suitability.

Contact us for more information about how we can help you serve your members!

Product	Absorbency	Description	Type	Mobility	Gender
Bladder Control Pads  Male Guard 	Light	<ul style="list-style-type: none"> • Disposable product designed to attach to form-fitting underwear. • Absorbs small amounts of urine leakage (fluid). • Bladder control pads are different from feminine pads which absorb blood. • Male Guard: Bladder control pad with a tapered end specifically designed for men. 	Urinary – Stress Incontinence	Mobile or Assistant Device	
Protective Underwear Pull-Ons 	Light - Moderate	<ul style="list-style-type: none"> • Applied similar to underwear. • Designed for moderate urinary incontinence, not a full void. • Not designed to hold bowel movements other than very small, formed stool. • Difficult to apply to someone confined to a bed. • Not recommended at night. • These are available in multiple absorbency levels but are never recommended for heavy urinary incontinence or bowel incontinence. 	Urinary – Stress, Mixed, or Urge Incontinence	Mobile or Assistant Device	

Product	Absorbency	Description	Type	Mobility	Gender
Briefs <i>Adult Diapers</i> 	Moderate - Heavy	<ul style="list-style-type: none"> Disposable product designed to absorb urine and help contain feces. Offers full front and back absorbency protection. Intended for anyone with nighttime urine or fecal incontinence. Recommended for bedbound and chairbound individuals, and those who are mobile but have bowel incontinence. Briefs are also available in multiple absorbency levels to fit the needs of each user. A brief is the only product appropriate for bowel incontinence. 	Urinary & Bowel - Overflow Incontinence	Mobile, Assisted or Wheelchair /Bedbound	
Booster Pad 	Light	<ul style="list-style-type: none"> Used with other incontinence products like pull-ons or briefs as an extra layer of protection to help boost absorbency. Consider a higher-absorbency product before resorting to a booster pad. 	Urinary	Mobile or Assistant Device	
Underpads 	Light	<ul style="list-style-type: none"> With a waterproof barrier on one side, these soft, thin pads help protect bedding, furniture, wheelchair seats, and other surfaces. Disposable and machine washable underpads available depending on insurance. 	Urinary & Bowel	Wheelchair /Bedbound	